BLOOD COLLECTION AND TRANSFUSION SYSTEMS

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KAWASUMI BLOOD BAG
QUADRUPLE450

CPD WHOLE BLOOD (HUMAN) Donor Number
Rh Type Content of CPD
For the collection of a blood

ABO REF
Date of collection USE BY
Expiration date LOT
Each 100ml of CPD solution contains:
Sodium citrate dihydrate 2.63g
Citric acid monohydrate 3.27g
Glucose, anhydrous 2.32g
Sodium dihydrogen phosphate dihydrate 251mg
Water for injection to 100ml

Non-pyrogenicity
Do not vent. - Do not use in case of any visible sign of hemolysis or deterioration.
Mix blood and anticoagulant at several times during collection. - Do not add any medicine.
Crossmatch before transfusion. - Inhibition set must have a filter.

TO BE STORED AT 4°C ± 2°C

CE 0123

KAWASUMI LABORATORIES, INC.
3-28-15, SHIBUYA-KU, TOKYO, JAPAN
MPS Medical Product Service GmbH
Borngasse 70, 35619 Braunfels, Germany

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Ultra Thin Wall Needle

The ultra thin wall needle maximizes blood flow, achieves faster blood collection and reduces discomfort during venipuncture. A unique blue colored film with a black line on the needle cover indicates if the needle cover has been removed or tampered with.

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Snap Tip

Our snap tip design facilitates smooth transfer from the collection bag. The distinctive shape enables the smooth flow of SAG-M solution into the collection bag.

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Tubing

The tubing is extremely flexible and highly resistant to kinking when it is bent.

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Detachable Port Protector

The protector can be detached completely from the bag which enables a clean transfer without contamination from the blood port.

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Top & Bottom System

(anticoagulant CPD, additive solution SAG-M)

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CE 0123

Needle Protector

Anti-needle stick protector covers donor needle upon removal from the vein to prevent accidental needle stick injuries.

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Sampling Port

Pre-attached vacuum tube holder that facilitates blood sampling collection.

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Sampling Pouch

The sampling pouch is used to collect the first blood flow from the donor to prevent contamination.

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Bag Material

A specially designed inner surface of the bag minimizes trauma to cellular components such as red cells and platelets.

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Rounded Corner

This shape assists in the smooth transfer of blood during separation and transfusion.

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Transfer Bag:

Ports are tamper evident. They feature a pull out tab, which guarantees that the seal has not been opened.

The design of the spike eliminates the possibility of puncturing the bag since the spike ruptures only the membrane of the port.

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Double Bag
Bolsa Doble

Whole Blood Transfusion Set:

The blood administration set has been designed to achieve a safe and effective blood transfusion. It has a 200-micron mesh filter, which traps blood aggregates and prevents blood clotting at the puncture site. Unlike the competition, Kawasumi offers a sharp Universal spike which enables easy puncturing of spike port. Many competitive products require clinicians to cut the spike port.

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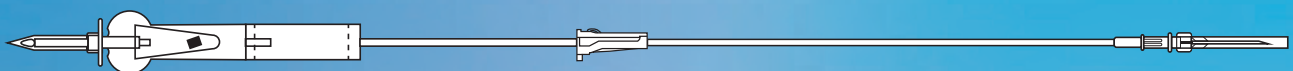
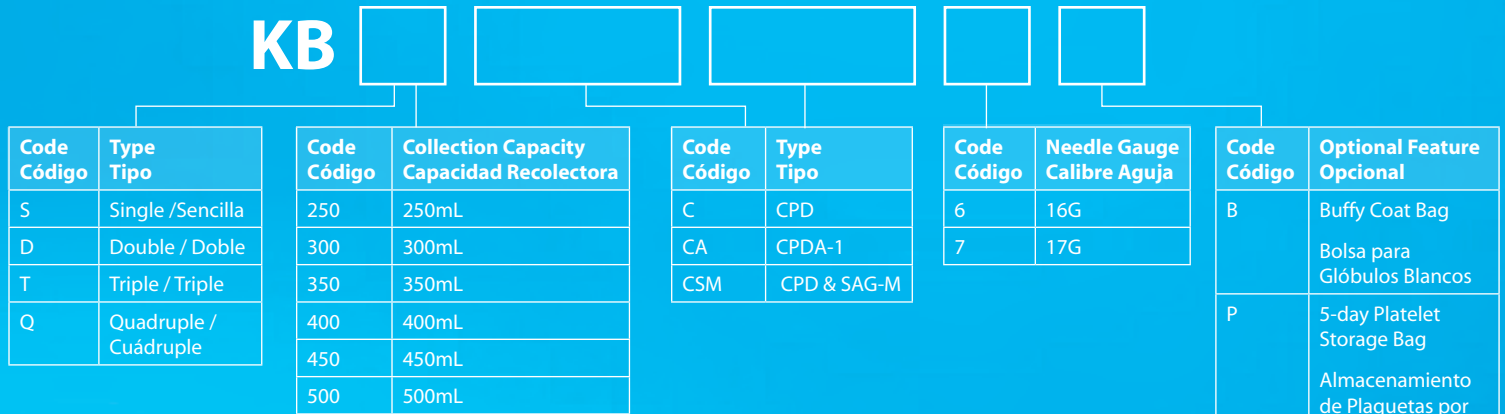
<Example>

Code Number KBT-450CSM6PY

This is Triple bag, 450 mL collection volume, filled with CPD-SAG/M solution, a pre-attached 16G donor needle, with a 5-day Platelet Storage Bag, Sampling Port and Sampling Pouch.

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CPD: 21-day Whole Blood & Red Cells Storage

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CPDA-1: 35-day Whole Blood & Red Cells Storage

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*450 mL bags with CPDA-1 solution and 450 or 500 mL bags with SAG-M solution available by special order.

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